Uriscreen™ Diagnostic Kit Uranotest®

GK

www.
uranovet.com

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EN

www.
uranovet.com

+information and technique demonstrative videos

Technical basis

Uranotest® Uriscreen™ is a rapid test for the detection of catalase(+) bacteria, and somatic cells (leukocytes) in the urine of dogs and cats, both key in urinary tract infections.

Uranotest® Uriscreen™ consists of a rack with 10 test tubes containing a reagent powder. When adding the urine sample together with hydrogen peroxide, if there is bacteriuria and/or the presence of somatic cells (leukocytes), white foam will be generated, indicating urinary tract infection.

Materials supplied

- 1 - Test tubes with cap
- 2 - H₂O₂ dropper bottle.
- 3 - Disposable pipettes.
- 4 - Booklet with instructions.

Precautions

- 1 - For veterinary use only. For in vitro use only. Keep out of reach of kids.
- 2 - Uranotest® Uriscreen™ contains a 10% hydrogen peroxide solution (H₂O₂) and a colored reactive powder that can stain and irritate. Do not heat or mix with flammable substances. Avoid contact with eyes, skin and clothing. In case of contact, rinse immediately with plenty of water.
- 3 - All samples must be considered as potentially infectious; wash and disinfect hands after handling.
- 4 - For an optimal result, strictly follow the instructions for use.
- 5 - Do not use the product or its components after the expiration date has passed.
- 6 - The quality of the components has been individually evaluated for each batch. Do not mix reagents or components from different lots.

Preservation and stability

Store in a cool, dry place between 10-28°C. Do not expose to a temperature >37°C.

DO NOT FREEZE. Expiration: 24 months.


Sample collection and preparation

Collect the urine sample in a clean container and test as soon as possible (ideally within 1 hour of collection to prevent growth of bacteria pathogens). If this cannot be done, the sample can be stored refrigerated for no more than 6 hours.

Instructions for use

- 1 - Transfer the urine to be analyzed to the supplied test tube using the disposable pipette. Add the urine sample, so that the added volume is contained between the minimum and maximum lines printed on the media (approximately 1.5-2 ml). Use a different test tube for each urine sample.
- 2 - Add 4 drops of 10% hydrogen peroxide (H₂O₂) solution to each tube of trial. Mix gently, so as not to foam, for five seconds. The solution will turn blue.
- 3 - After 1-2 minutes, observe the formation of foam and check the results. If the result is positive, foam will form on the surface of the liquid. If the result is negative, foam will not form.

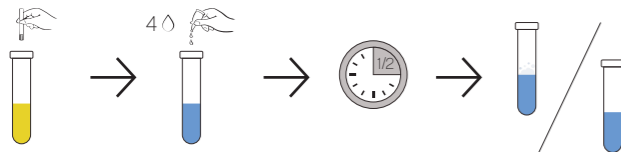Fig. 1

Interpreting results

1 - Negative result

No foam is generated or the generated foam ring remains incomplete after two minutes. A negative result indicates that there is no presence of catalase + bacteria.


Fig. 2

2 - Positive result

Sufficient foam will form to form a complete and continuous ring or layer on the surface of the liquid. Foaming indicates the presence of catalase + bacteria and/or white blood cells indicative of urinary tract infection. In this case, it is recommended to send the sample to the laboratory to determine the bacteria causing the infection and perform the corresponding antibiogram to choose the most effective antibiotic in each case.

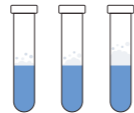Fig. 3

Limitations of the technique

Although the Uranotest® Uriscreen™ Diagnostic Kit has a high sensitivity and specificity, a small incidence of false positive or negative results cannot be ruled out. As with any other laboratory procedure, the definitive clinical diagnosis should not be based solely on the performance of a test, but must be the set of a series of clinical and laboratory findings. In case of doubt, repeat the test and/or contrast with other diagnostic methods.

Uranotest® Uriscreen™ does not detect catalase negative organisms, and false negative results may occur if the bacteria are catalase negative and the level of white blood cells is not high enough.

False-negative results could also be related to frequent urination (urine not held in the bladder for at least four hours), since, in this case, there may not be enough bacteria in the urine to produce catalase activity.

Other cells present in the urine can also cause false positive results.

Kit diagnóstico Uranotest® Uriscreen™

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PT

www.
uranovet.com

+information and technique demonstrative videos

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