Kit diagnóstico Parvo-Corona-Giardia Uranotest

For veterinary use only

EN

www.
uranovet.com

+information and technique demonstrative videos

Technical basis

URANOTEST PARVO-CORONA-GIARDIA diagnostic kit is based on the immunochromatographic technique and is designed for the qualitative detection of *Canine Parvovirus* (CPV), *Canine Coronavirus* (CCV) and *Giardia* antigens in dog faeces.

The test has a triple structure; it contains three single tests: a strip for CPV antigen detection, a strip for CCV antigen detection and a strip for Giardia detection. Each test consists of several overlapped membranes. On one of the membranes, there are a test line (T line) and control line (C line). The lines are not visible before applying the sample. After applying the sample in the appropriate sample well, migration begins by capillary action through the membrane. If the result is negative, one purple colour band appears in the C area. This line, called control line, always appears, as it is a control line indicating that the test has successfully performed. If the test result is positive, in addition to the control line, a second line will form in the test area (Test line).

Materials supplied

- 1 - Triple test devices individually packaged in aluminium pouch.
- 2 - Tubes with buffer solution for sample dilution.
- 3 - Swabs for sample collection.
- 4 - Disposable pipettes.
- 5 - Instructions for use.

Precautions

- 1 - For veterinary use only.
- 2 - Wear disposable gloves when handling the samples. All samples should be treated as potentially infectious. Wash and disinfect hands after handling. Avoid aerosol formation when dispensing the sample.
- 3 - To obtain good results, it is important to add the correct sample volume.
- 4 - Open the device just before use.
- 5 - All reagents must be at room temperature before performing the test.
- 6 - Do not use the test if the envelope is damaged or broken.
- 7 - Do not re-use.
- 8 - Do not use reagents after the expiry date.
- 9 - The quality of each component of the kit has been individually assessed for each batch. Do not mix components or reagents from kits with different batch numbers.

Preservation and stability

The kit must be stored at a temperature between 2 and 30°C. Under these conditions, we can guarantee the stability until the expiry date printed on the box and on the individual pouch.


The kit has been developed to be stored at room temperature. Although it also can be stored in the refrigerator, we recommend store it at room temperature to avoid the need to wait for reagents to reach the room temperature.

DO NOT FREEZE. Do not expose to direct sunlight.

Sample collection

The sample may be collected directly from the rectus using the provided swab. The sufficient amount of sample to carry out properly the test is shown in the picture (see figure 1).

Introduce the swab into the tube containing the diluent and press it against the tube walls in order to release the sample that could contain the virus. Shake the tube in order to ensure a good homogenisation.

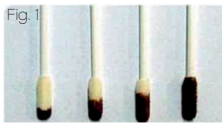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